

Door Connectors

Through the door – quick and easy



Room covering systems with Ergolet's door connector.

Primary application

Home care
Nursing homes
Hospital

Door connectors are used for patient transfer from room to room. Ergolet's simple and reliable mechanism is used for transferring patients from room to room, e.g. bedroom to bathroom, thereby reducing the number of total lifts required. Fewer lifts creates a better working environment for the carer and safer transfer for the patient.

Ergolet's door connector, as a wall mounted solution, is the preferred choice by architects and building contractors, because of its elegant and compact design. The door connector is suitable for most wall widths. And because it is mounted very close to the wall surface taking up little space in the room, the desired room covering area is exploited optimally.

Ergolet's door connector is only 300 mm wide and does not require costly changes to the building's electrical system, as it operated manually.

Locking and unlocking the door connector is simply done by lightly pulling on the cord. When the door connector is unlocked and the traverse is aligned with the mechanism, the system locks automatically, ready for room to room transfer. To unlock pull on the cord.

Key standard functions

Compact and discreet
Only 300 mm wide
Flexible for fixed and traverse systems
Manual operation





1. Pull the cord to activate door connector. Slide the traverse towards the door connector.



2. When the traverse is aligned with the door connector, it will automatically lock into position with a click sound ready for room to room transfer.




3. When the ceiling hoist has been moved into the other room, pull on the cord to unlock the mechanism and to move the traverse.



Traverse track 135 mm
Connector 135 mm
Traverse track 135 mm



Fixed 135 mm
Traverse track 135 mm

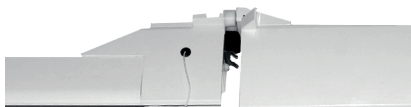

Fixed track 75 mm
Traverse track 135 mm

PRODUCT HIGHLIGHTS

- Manual operation – no costly electrical installations required
- Traverse is locked and unlocked by simply pulling the cord
- Compact and discreet
- Several models to suit individual configurations
- Fail safe mechanical features


Version	Item-number
Traverse track 135 mm - Connector 135 mm - Traverse track 135 mm	016-01080
Fixed track 135 mm - Traverse track 135 mm	016-01081
Fixed track 75 mm - Traverse track 135 mm	016-01073




Unlocking the door connector happens easily by pulling the cord. Pull the cord again to lock the mechanism.

TECHNICAL SPECIFICATION


	Connector 135mm	Fixed track 75mm
Length	300 mm	220 mm
Height	145 mm	145 mm
Width	100 mm	100 mm

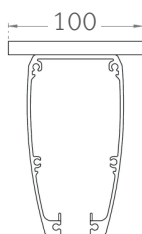

Drawings refer to connector 135 mm

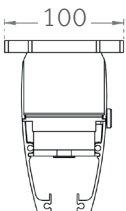

Drawings refer to fixed track 75 mm

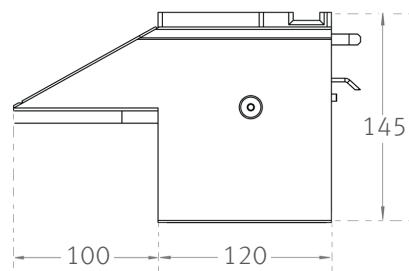

Consult the Technical manual for further specifications. All figures and measurements are for guidance only and may differ in actual situations. Ergolet reserves the right to changes without notice.

Product photos may include accessories. Consult your dealer.



date 02.04.2014 - ver. 4

Ergolet
Taarnborgvej 12C
4220 Korsoer
Denmark
tel. +45 70 27 37 20
e-mail: ergolet@ergolet.dk



www.ergolet.com