

EKAMOVE - QUICK START GUIDE




This Quick Start Guide is not intended to replace the full user instructions. Please read the full user instructions before using the Ekamove.




Set Up

DO NOT USE THE EKAMOVE SYSTEM WITHOUT SIDE RAILS FITTED TO THE BED, IRRESPECTIVE OF THE DEGREE OF TILT REQUIRED. WE RECOMMEND THAT SIDE RAILS ARE ALSO FITTED WITH BUMPER PADS.


1. Strap the Ekamove to the bed frame with sensors level with patients shoulders (On profiling beds fit straps to moving parts of the frame only).
2. Fit the control unit to the foot of the bed and clamp in place.
3. Run sensor lead and air hoses to the control unit ensuring that they avoid any pinching or trapping points and connect to the underside of the unit.
4. Switch the control unit 'On'.

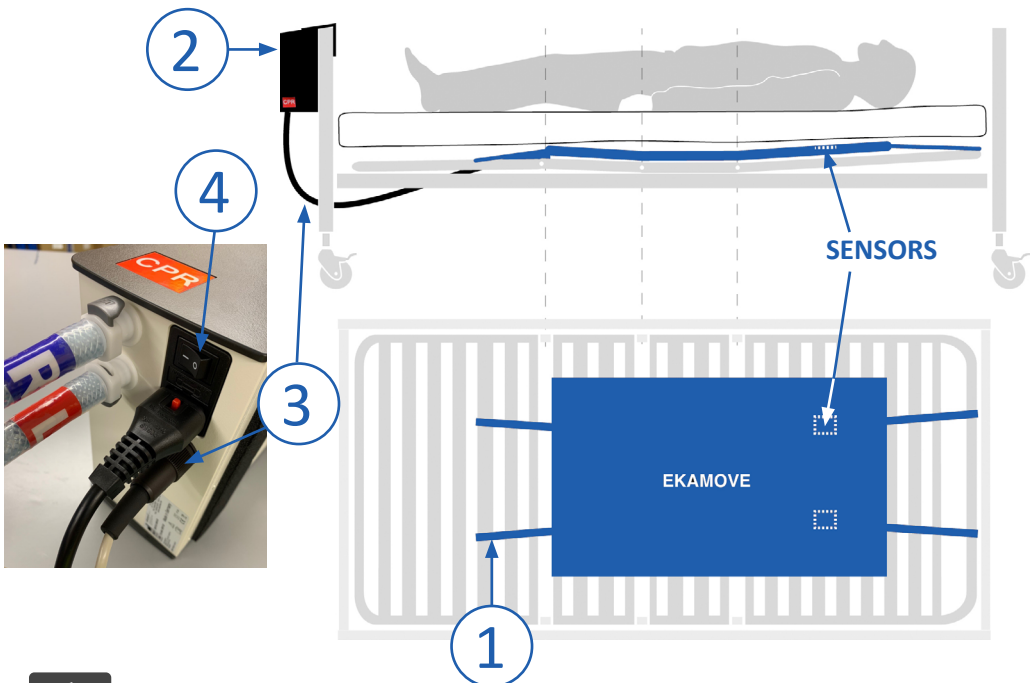

Programming

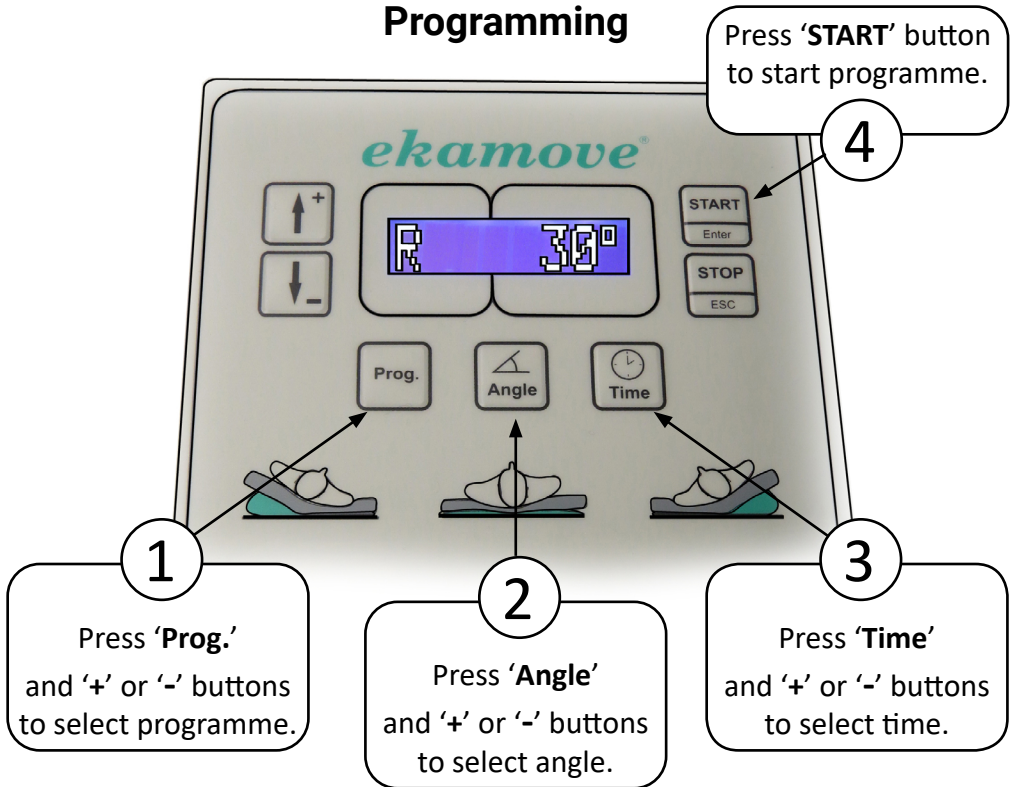

IF ERROR CODE - **ERR YPOS** IS DISPLAYED - SIMPLY LOWER THE BED BACKREST UNTIL THE ALARM STOPS (under 25 degrees).

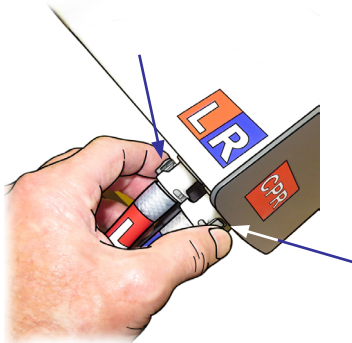

NOTE: The air hoses can be released to vent air quickly by pressing together the two grey lugs where they connect to the control unit.

WinnCare

Presteigne, Powys, Wales, LD8 2UF, UK

Tel: +44(0)1544 267674

Email: winnCare@winnCare.uk

Web: www.mangarhealth.com

MI0444 Issue 7