WIN  CARE

MANGAR

AIRFLO DUO COMPRESSOR

USER INSTRUCTIONS

ENGLISH

DEUTSCH

FRANCAIS

Contents

Intended Use	Pg 2
Warnings	Pg 2
Battery Status Indicator	Pg 3
Battery Charging	Pg 4
Mains Power Adaptor (<i>optional</i>)	Pg 5
Operation	Pg 5
Cleaning & Storage	Pg 6
Maintenance	Pg 6
Trouble shooting guide	Pg 7
Specifications	Pg 7
Symbols used on Airflo Duo and Charger	Pg 8
Symbols used on packaging	Pg 9
Warranty	Pg 9
Declaration of Conformity	Pg 26

Intended Use

The Mangar Airflo Duo is a low pressure air compressor, which has been designed to inflate the Mangar 'Rhino' and 'Swift' pneumatic lifting equipment.

Applied Parts (Accessories) for the Airflo Duo:

- Mangar Rhino (Supine Lifting Cushion)
- Mangar Swift (Supine Transfer Slide)

The Rhino and Swift require two or more people to operate. The Airflo Duo must only be used by professionals trained in its use.

The Mangar Rhino Lifting Cushion is designed to lift a person in the supine position from the floor. When inflated, the Rhino lifts the patient to the height of a bed or stretcher.

The Mangar Swift Transfer Slide enables healthcare workers to transfer and reposition patients in the supine position with minimal moving and handling force needed.

Further guidance on operation can be found in the individual accessory User Instructions. The Airflo Duo compressor and applied parts are intended for continual re-use as required and no special actions or operations are needed to achieve this.

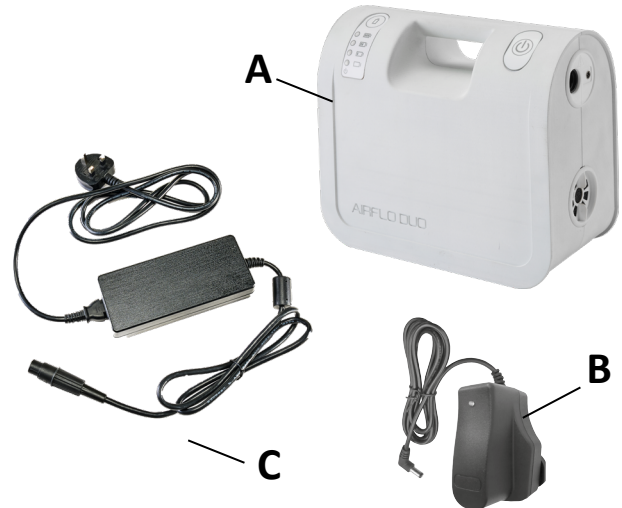
WARNINGS

- Be sure that you have read and understood these instructions and those provided with any other Winnicare product, before operating the equipment.
- To avoid the risk of electric shock, NEVER recharge the battery in the bathroom, outdoors or in a damp environment.
- To avoid the risk of electric shock, do not use the optional mains power adaptor in the bathroom, outdoors or in a damp environment.
- Avoid the use of the Airflo Duo in wet conditions.
- Always disconnect from the battery charger before cleaning.
- Use only the supplied battery charger.
- Ensure that when in use, the Airflo Duo and its connected hose are laid out in such a way as to avoid any risk of tripping or strangulation.
- Ensure that children and pets do not interfere with the safe operation of the Airflo Duo and inflatable product.
- Keep the Airflo Duo and inflatable product away from sources of heat.
- Disconnect the Airflo Duo compressor from the inflatable product before transporting so as to avoid accidental inflation.
- The Airflo Duo should only be used for its stated purpose and should not be modified in any way.
- The optional mains power adaptor (The Mean Well Enterprises AC/DC Switching Adaptor Model GST160A24-R7B) is specific to the Airflo Duo and is the only approved unit to be used for this purpose.
- **WARNING:** This product and associated packaging does not contain any materials known to cause an allergic reaction. **Please contact customerservices@winnicare.uk if you have specific concerns.**

Standard Product Contents

- 1 x Airflo Duo compressor (A) fitted with a 25.2v - 6200mAh Li-Ion battery pack.
- 1 x Battery charger (with mains adaptors) (B).
- 1 x User Instructions.
- 1 x Mains Power Adaptor. (C)

Power Button

When the 'Power' button is pressed, the compressor will turn on once connected will begin inflating the inflatable product. The battery state indicator will show the battery capacity at the start of operation.

Battery State Indicator

When the Battery State Indicator button is pressed, the lights indicate how much charge remains in the battery.

RED
Recharge
or connect mains
power

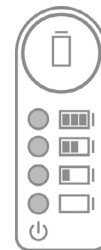
1 GREEN
OK to Use

2 GREEN
OK to Use

3 GREEN
OK to Use

4 GREEN
OK to Use

WARNING: When the red indicator light appears, either continue to use with the mains power adaptor connected, or place on charge. Running the Airflo Duo on RED until the battery is exhausted will damage the battery and risk being unable to complete a full lift or transfer.

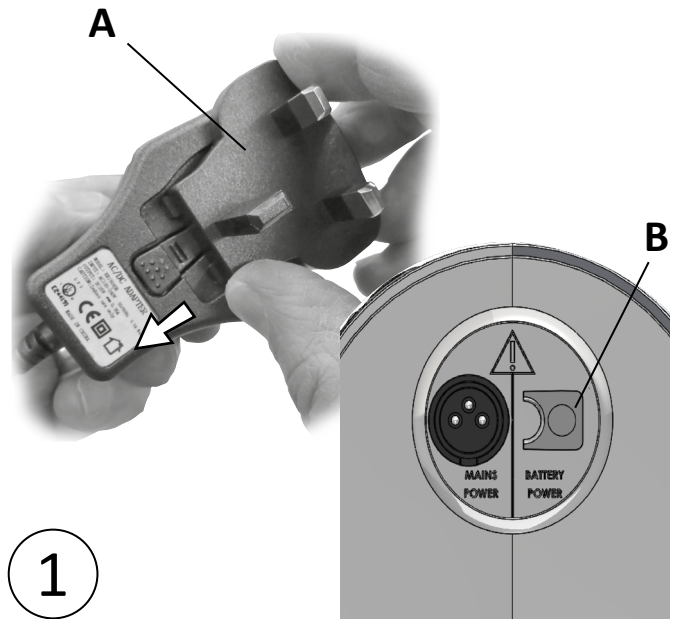
When the compressor is running, the battery status indicator lights will only indicate the state of battery charge as it was at the start of the lift.

IMPORTANT: To maximize the lifting performance of the battery, fully recharge the battery after use.

Battery Charging - fig. ①

Fully charge the battery before first use.

- Check the battery charger casing and lead for signs of damage and if damaged DO NOT USE.
- Fit the appropriate mains plug adaptor to the battery charger (A).
- Plug the battery charger into a mains power socket and connect the charger output lead to the Airflo Duo charging socket (B).

①

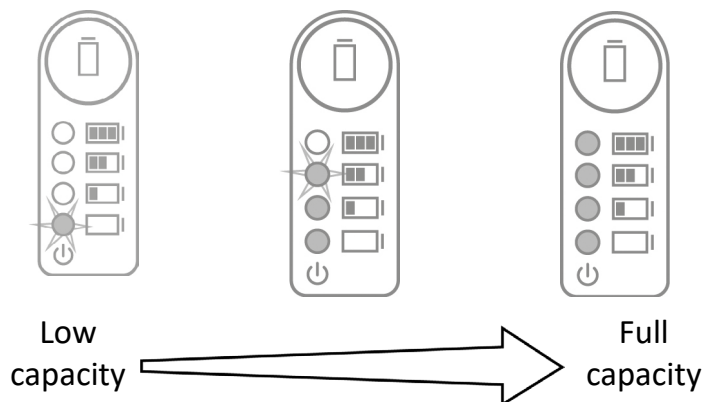
NOTE: On first charge or when fitting a new battery it should be fully charged before use. On subsequent charges the Airflo Duo measures the battery capacity and charges as required. As the battery charges, the indicator lights will incrementally increase to display battery capacity.

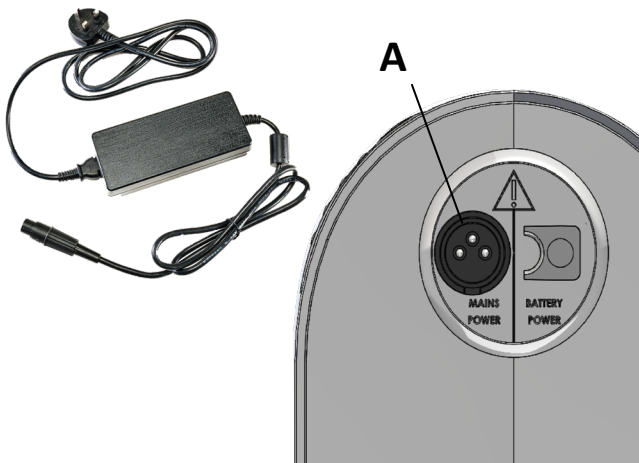
When the battery is fully charged:

- The compressor will move into an energy saving mode with the top LED flashing every 6 seconds and the other 3 will be off.

NOTE: The compressor will not run whilst the charger is connected.

To ensure optimum performance, it is recommended that the Airflo Duo is left continuously on charge when not in use. The batteries cannot be overcharged using this charger.



2

Mains Power Adaptor- fig. ②

A mains power adaptor is available to power the Airflo Duo directly from a mains power socket. The adaptor is specific to the Airflo Duo and is the only approved unit to be used for this purpose. Connect the adaptor between a mains power socket and the adaptor inlet socket on the end of the Airflo Duo (A).

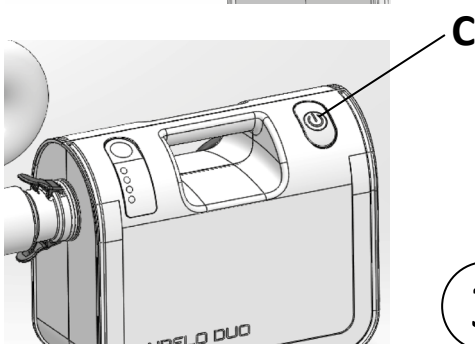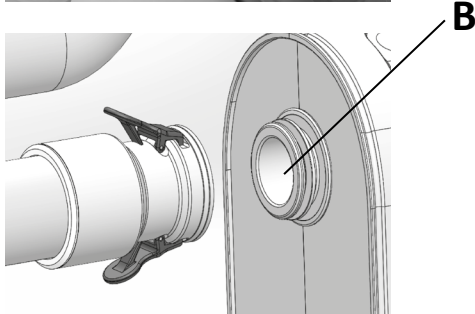NOTE: This adaptor, has a non-standard output connector and powers the Airflo Duo directly, it will not charge the battery. The only way to charge the battery is to use the charger which was supplied with the Airflo Duo. With the power adaptor connected, the Airflo Duo behaves in the same way as if it was running on the battery, the only functional difference is that the battery indicator will show bottom LED orange whilst the Airflo is running.

3

Operation - fig. ③

Disconnect the Airflo Duo from the battery charger.

- Connect the hose (A) to the valve on the inflatable product. When using to inflate the Mangar Rhino, ensure you always inflate the sections in order from 1 through to 4. Please refer to the separate Rhino User Instructions for full detail.
- Connect the hose from the inflatable to the Airflo Duo air outlet socket (B). The connector will 'click' into place when firmly pushed onto the socket. Note: Either end of the hose is compatible with the inflatables and the Airflo Duo.
- To switch on, press the 'Power' button (C). The compressor will then begin to inflate the product.
- If using to inflate the Rhino, when full inflation is reached on each tier, all 4 lights will flash.
- Detach the hose from the inflatable before pressing the power button to switch the Airflo Duo off. The indicator lights will go out.

Cleaning

Unless contamination is suspected, periodic cleaning is all that is recommended. The Airflo Duo should be cleaned/disinfected as follows:

- Unplug the Airflo Duo from the battery charger. Clean the external surfaces using a non abrasive damp cloth with warm water. Clinell Universal wipes or similar can be used. Wipe dry thoroughly with a dry towel before reconnecting to the battery charger. Do not allow moisture to enter any part of the compressor.
- Do not use solvent-based products.

Disinfecting/Sterilising

- Disinfection – Pre-clean the Airflo compressor by wiping with a damp cloth and then disinfect by wiping with a 1:10 solution of household liquid bleach and water- 1 part bleach to 10 parts cold water (10,000ppm (1%) Sodium Hypochlorite solution). Do not allow moisture to enter any part of the compressor. Allow to air dry. In usual circumstances this product does not require sterilisation for reuse.

Storage

- When not in use, store in a dry place together with these instructions. WinnCare recommends you ensure the Airflo Duo is fully charged before storing. To maintain the Airflo Duo battery when being stored for long periods, ensure it is fully charged every 3 months.

Maintenance

The Airflo Duo has been designed to be maintenance free and apart from replacing the battery, the Airflo Duo contains no other user serviceable parts.

Battery replacement:

- Battery replacement should be carried out by a professional. Please contact your supplier if you require a battery replacement.

When it becomes apparent that the Airflo Duo is producing fewer than expected lifting cycles or the maximum pressure is no longer being reached, the battery is probably reaching the end of its working life and should be replaced.

If a fault should develop or if you require assistance in setting up using or maintaining the equipment, please contact your supplier.

Disposal

When the Airflo Duo has reached the end of its working life, remove the battery for safe disposal and dispose of the Airflo Duo using your organisation's recycling/disposal system.

Trouble shooting guide


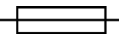
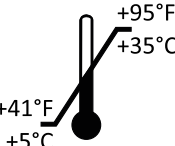
Problem	Remedy
<ul style="list-style-type: none"> Compressor fails to turn on 	<ul style="list-style-type: none"> Ensure that the Airflo Duo has been switched on and is charged or connected to mains power. Ensure the charger is disconnected from the Airflo.
<ul style="list-style-type: none"> No Battery status indicator Lights 	<ul style="list-style-type: none"> Ensure the Airflo Duo is charged.
<ul style="list-style-type: none"> Bottom Battery status indicator solid Red Light 	<ul style="list-style-type: none"> Low Battery Capacity, recharge.
<ul style="list-style-type: none"> Bottom Battery status indicator Flashing Red every 2 seconds 	<ul style="list-style-type: none"> Ensure the correct mains power adaptor is connected.
<ul style="list-style-type: none"> Bottom Battery status indicator flashing Red every ½ second 	<ul style="list-style-type: none"> Battery Voltage low – Re charge. If occurs repeatedly replace the battery.
<ul style="list-style-type: none"> Compressor causes electrical interference with another piece of electrical equipment 	<ul style="list-style-type: none"> Move either the compressor or other item further away from each other until the interference stops.

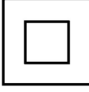For further assistance with resolving a problem please contact our Customer Service Department by calling +44(0)1544 267674 or email customerservices@winncare.uk

Specifications

Airflo Duo	
Weight:	2.8Kg
Output pressure:	0.069 Bar (1 psi)
Batteries:	1 x 25.2v- 6200mAh Li-Ion Airflo Duo Battery Pack
Battery Life:	500 charging cycles (when used in accordance with battery charging instructions). For optimal performance, WinnCare recommends replacing batteries every 24-36 months dependent on use.
Environmental protection: IP41 (Protection from entry by solid objects with a diameter or thickness greater than 1.0mm & dripping water)	
Charger: Model number - SP3009SRM	Mains Power Adaptor:
Input:	<i>Meanwell Switching Adaptor</i> - GST160A24-R7B
	Input:
	AC 100v - 240v,
	50-60Hz, 0.35A Max
Output:	DC 30v, 0.45A
Class II Double insulated equipment.	Output:
	AC 100v - 240v,
	50-60Hz, 2A
	Output:
	24V 6.67A, 160W

Symbols used on Airflo Duo and Charger labels

<p> Attention consult instructions</p> <p> Type BF equipment (protection against electric shock)</p> <p> Alternating current (AC)</p> <p> Fuse (with rating)</p> <p> Direct current (DC)</p> <p> Max/Min operating environment temperature</p> <p> For indoor use only. Do Not connect to mains power in a bathroom or outdoors.</p> <p> Battery Status Indicator Button</p>	<p> Class II Double Insulated equipment</p> <p> Dangerous voltage</p> <p> Recycle</p> <p> Do not dispose of in household waste</p> <p> Date of manufacture</p> <p> Power Button</p>
---	--

Symbols used on packaging

	Keep packaging dry		Do not dispose of in household waste
	Handle with care	93% RH 15% RH	Max/Min humidity when transporting or in storage
	Net & gross weights	700 mBar 1060 mBar	Max/Min pressure when transporting or in storage
5 	Stack no more than five high	+70° C -25° C	Max/Min temperature when transporting or in storage
	Recycle packaging		

Warranty

The warranty is given against defects in material and workmanship and comprises piece, parts and labour. The warranty is not valid in the case of fair wear and tear, wilful damage, common neglect, misuse or unauthorised alteration or repair by unqualified persons.

The following statements do not affect the consumer's statutory rights. This product is sold on the understanding that in the event of any defect of manufacture or material appearing within the warranty period, the defect will be rectified free of charge providing that:

- Reasonable evidence is provided (e.g. purchase invoice, delivery note) that the claim is being made within the warranty period.
- The defect is not attributable to accidental damage (either in transit or otherwise), misuse or unauthorised repair.

Airflo Duo serial number:

Date of purchase:

Invoice/Delivery note number:

WinnCare recommends that your Airflo Duo is inspected annually by either your supplier or other suitably qualified person. If required, WinnCare can provide the above inspection within an annual service contract (UK only). Please contact our Customer Service Department for further details on Freephone 0800 2800 485.

Mangar International, trading as WinnCare, reserves the right to alter product specifications and/or any of the information contained within this document without notice.

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WinnCare is a trading name of Mangar International Ltd.
Mangar International Ltd, Presteigne, Powys,