

Functionality, tailored to your requirements



WINNCARE
N o r d i c

Various models and sizes / New: Disposable sling

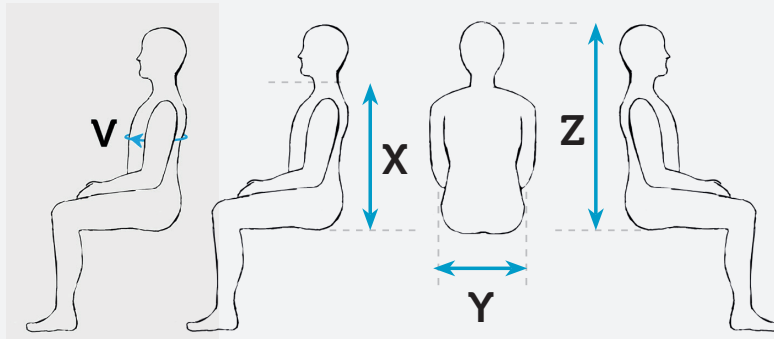


Table of contents

4-5	Sling information
6-9	Universal Basic (4 models)
10-11	Universal Hammock (2 models)
12-13	Universal Amputee (2 models)
14-15	Universal Comfort (4 models)
16	Disposable sling
17	Toilet sling
17	Low Back
17	Leg sling
18	Walking sling
19	Standing sling

Sling information

Sling size chart



All UNIVERSAL slings *without* head support, (HS) also TOILET and LOW BACK slings

	XXS	XS	S	M	L	XL	XXL
Y (hip measurement)	-	37 cm	42 cm	48 cm	52 cm	68 cm	78 cm
X (measurement from neck to lower back)	-	50 cm	55 cm	66 cm	75 cm	75 cm	75 cm

All UNIVERSAL slings *with* head support (HS)

Y (hip measurement)	-	37 cm	42 cm	48 cm	52 cm	68 cm	78 cm
Z (measurement from head to lower back)	-	75 cm	80 cm	91 cm	100 cm	100 cm	100 cm

STANDING sling

V (chest measurement)	-	62 – 80 cm	66 – 90 cm	80 – 100 cm	100 – 126 cm	110 – 132 cm	120 – 140 cm
------------------------------	---	------------	------------	-------------	--------------	--------------	--------------

WALKING sling


V (chest measurement)	62 – 75 cm	76 – 84 cm	85 – 92 cm	93 – 105 cm	106 - 114 cm	116 - 130 cm	-
------------------------------	------------	------------	------------	-------------	--------------	--------------	---

Spreader bar models



Sling size guide

Explanation and colour code are presented on the size chart on page 4. The picture below shows red colour code, which means that the sling is in size S (Small). Colour code is located on the size indicator above sling label (as shown on the picture).

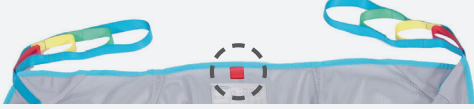

Sling attachment on patented hook system

- Pull upwards to unlock sling hook (1)
- A minimum of downward pressure locks the sling hook (2)



Material



Polyester

Extra durable and easy to clean.



Polyester mesh

Suitable especially for bathing.



Nonwoven

For disposable slings. See page 16.

i Overview of sling models

Category	Name	Polyester w/o HS	Polyester with HS	Polyester mesh w/o HS	Polyester mesh with HS
Full sling	Universal Amputee	●	●	○	○
Full sling	Universal Basic	●	●	●	●
Full sling	Universal Comfort	●	●	●	●
Full sling	Universal Hammock	○	○	●	●
Half sling	Low Back	●	○	○	○
Half sling	Toilet	●	○	○	○
Standing sling	Standing	●	○	○	○
Standing sling	Standing 2-Belt	●	○	○	○
Walking sling	Walking	●	○	○	○

Universal Basic Mesh

Head support



- Model**
- Universal Basic HS mesh XS
 - Universal Basic HS mesh S
 - Universal Basic HS mesh M
 - Universal Basic HS mesh L
 - Universal Basic HS mesh XL
 - Universal Basic HS mesh XXL

- Item no.**
- 009-09833
 - 009-09834
 - 009-09835
 - 009-09836
 - 009-09837
 - 009-09838


When to use

The sling with head support is for users, who need extra support around the head. The sling provides perfect support around the thighs, upper back, and upper body giving the user a feeling of comfort and safety.

Used for transfers from a sitting or lying position.
For users with lack of torso stability.

Sling in polyester mesh is suitable for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg
Type	Full sling
Material type	Polyester mesh

Washing	
Spreader bar	2, 3 and 4 point
Sizes	XS - XXL



Scan QR code for **video** -
when to use Universal
Basic sling



Choose a
Universal sling
that fits your
needs best

Universal Basic Mesh



Model	Item no.
● Universal Basic mesh XS	009-09813
● Universal Basic mesh S	009-09814
● Universal Basic mesh M	009-09815
● Universal Basic mesh L	009-09816
● Universal Basic mesh XL	009-09817
● Universal Basic mesh XXL	009-09818

i When to use

The sling without head support is for users, who have a good head control. The sling provides perfect support around the thighs and upper back, thighs, upper back, and upper body giving the user a feeling of comfort and safety.

Used for transfers from a sitting or lying position.
For users with lack of torso stability.

Sling in polyester mesh is suitable for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester mesh	Sizes	XS - XXL

Universal Basic Head support



Model	Item no.
● Universal Basic HS XS	009-09823
● Universal Basic HS S	009-09824
● Universal Basic HS M	009-09825
● Universal Basic HS L	009-09826
● Universal Basic HS XL	009-09827
● Universal Basic HS XXL	009-09828

i When to use

The sling with head support is for users, who need extra support around the head. The sling provides perfect support around the thighs, upper back, and upper body giving the user a feeling of comfort and safety.

Used for transfers from a sitting or lying position.
For users with lack of torso stability.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL

Universal Basic



Model	Item no.
● Universal Basic XS	009-09803
● Universal Basic S	009-09804
● Universal Basic M	009-09805
● Universal Basic L	009-09806
● Universal Basic XL	009-09807
● Universal Basic XXL	009-09808

i When to use

The sling without head support is for users, who have good head control. The sling provides perfect support around the thighs and upper back, thighs, upper back, and upper body giving feeling of comfort and safety.

Used for transfers from a sitting or lying position.
For users with lack of torso stability.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL



// Practical and comfortable

Universal Basic is the perfect sling choice for extra support around the thighs and body. The sling has excellent seating comfort and provides good upright posture ensuring correct user positioning.

The model presented in the picture is the Universal Basic.



Universal Hammock

Mesh Head support



Model

- Universal Hammock HS mesh XS
- Universal Hammock HS mesh S
- Universal Hammock HS mesh M
- Universal Hammock HS mesh L
- Universal Hammock HS mesh XL
- Universal Hammock HS mesh XXL

Item no.

- 009-09683
- 009-09684
- 009-09685
- 009-09686
- 009-09687
- 009-09688

When to use

The sling with head support is designed for users, who need extra support around the upper body and head.

The sling provides excellent full body support, which gives the user a feeling of safety during transfers.

Used for less active users, who are in a seated position for longer periods. Used for transfers from a lying position.

The sling is made of polyester mesh, which is quick drying, porous and perfect for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg
Type	Full sling
Material type	Polyester mesh

Washing	
Spreader bar	2, 3 and 4 point
Sizes	XS - XXL



Scan QR code for **video** - when to use Universal Hammock sling



Universal Hammock Mesh



Model	Item no.
● Universal Hammock mesh XS	009-09673
● Universal Hammock mesh S	009-09674
● Universal Hammock mesh M	009-09675
● Universal Hammock mesh L	009-09676
● Universal Hammock mesh XL	009-09677
● Universal Hammock mesh XXL	009-09678


When to use

The sling without head support is designed for users, who have a good head control.

The sling provides excellent full body support, which gives the user a feeling of safety during transfers.

Used for less active users, who are in a seated position for longer periods. Used for transfers from a lying position.

The sling is made of polyester mesh, which is quick drying, porous and perfect for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester mesh	Sizes	XS - XXL



Scan QR code for **video** -
when to use Universal
Amputee sling



“ Stability and Safety

Universal Amputee sling has individual split leg straps: this special solution means that the leg straps are not crossed. The sling also has an extra soft padded cuff, which can be fastened around the amputee stump.

Universal Amputee Head support



Model	Item no.
● Universal Amputee HS XS	009-09753
● Universal Amputee HS S	009-09754
● Universal Amputee HS M	009-09755
● Universal Amputee HS L	009-09756
● Universal Amputee HS XL	009-09757
● Universal Amputee HS XXL	009-09758

When to use

The sling with head support is for users, who need extra support around the head.

The sling has extra support around the thighs and pelvis. The leg straps have a large padded surface, which reduces the pressure on the entire femur.

Used for single or double amputees.
Used for transfers from a sitting or lying position.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL



Universal Amputee




Model	Item no.
● Universal Amputee XS	009-09743
● Universal Amputee S	009-09744
● Universal Amputee M	009-09745
● Universal Amputee L	009-09746
● Universal Amputee XL	009-09747
● Universal Amputee XXL	009-09748

When to use

The sling without head support is for users, who need have good head control.

The sling has extra support around the thighs and pelvis. The leg straps have a large padded surface, which reduces the pressure on the entire femur.

Used for single or double amputees.
Used for transfers from a sitting or lying position.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL



Universal Comfort Mesh

Head support



Model

- Universal Comfort HS mesh XS
- Universal Comfort HS mesh S
- Universal Comfort HS mesh M
- Universal Comfort HS mesh L
- Universal Comfort HS mesh XL
- Universal Comfort HS mesh XXL

Item no.

- 009-09773
- 009-09774
- 009-09775
- 009-09776
- 009-09777
- 009-09778



When to use

The sling with head support is for users, who need extra support around the head.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

Sling in polyester mesh is suitable for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	 
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester mesh	Sizes	XS - XL



Scan QR code for **video** - when to use Universal Comfort sling



Universal Comfort Mesh



Model	Item no.
● Universal Comfort mesh XS	009-09853
● Universal Comfort mesh S	009-09854
● Universal Comfort mesh M	009-09855
● Universal Comfort mesh L	009-09856
● Universal Comfort mesh XL	009-09857
● Universal Comfort mesh XXL	009-09858

i When to use

The sling without head support is for users, who have a good head control.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

Sling in polyester mesh is suitable for bathing.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2 point
Material type	Polyester mesh	Sizes	XS - XXL

Universal Comfort Head support



Model	Item no.
● Universal Comfort HS XS	009-09763
● Universal Comfort HS S	009-09764
● Universal Comfort HS M	009-09765
● Universal Comfort HS L	009-09766
● Universal Comfort HS XL	009-09767
● Universal Comfort HS XXL	009-09768

i When to use

The sling with head support is for users, who need extra support around the head.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL

Universal Comfort



Model	Item no.
● Universal Comfort XS	009-09843
● Universal Comfort S	009-09844
● Universal Comfort M	009-09845
● Universal Comfort L	009-09846
● Universal Comfort XL	009-09847
● Universal Comfort XXL	009-09848

i When to use

The sling without head support is for users, who have a good head control.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

SWL	XS-L: 275 kg XL-XXL: 400 kg	Washing	
Type	Full sling	Spreader bar	2 point
Material type	Polyester	Sizes	XS - XXL

Universal Disposable Comfort Sling



Model	Item no. nr.
• Comfort disposable sling S (10 pcs.)	008-04104
• Comfort disposable sling M (10 pcs.)	008-04105
• Comfort disposable sling L (10 pcs.)	008-04106

When to use

Perfect for situations where a high hygienic standard is needed, e.g. if there is a risk of infection.

The sling without head support is for users, who have a good head control.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

SWL	275 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Materiale type	Nonwoven	Sizes	S -M-L

Universal Disposable Comfort Sling Head support



Model	Item no. nr.
• Comfort disposable sling HSS (10 pcs.)	008-04114
• Comfort disposable sling HS M (10 pcs.)	008-04115
• Comfort disposable sling HS L (10 pcs.)	008-04116

When to use

Perfect for situations where a high hygienic standard is needed, e.g. if there is a risk of infection.

The sling with head support is for users, who need extra support around the head.

The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.

For users with a degree of physical disability or other special needs. Used for transfers from a sitting or lying position.

SWL	275 kg	Washing	
Type	Full sling	Spreader bar	2, 3 and 4 point
Materiale type	Nonwoven	Sizes	S -M-L



Wash symbol with indicator

Warns you if the sling is damp or wet and therefore needs to be destroyed.

Once the sling is wet or damp, this warning will appear: "DO NOT USE. DESTROY"

Toilet



Scan QR code for video - when to use Toilet sling



Model	Item no.
● Toilet sling XS	009-09543
● Toilet sling S	009-09544
● Toilet sling M	009-09545
● Toilet sling L	009-09546
● Toilet sling XL	009-09547
● Toilet sling XXL	009-09548

When to use

The sling ensures a good upright posture that facilitates the correct positioning of the user.

The sling has a large opening around the lower body, which makes toileting and the changing of clothes easier.

Used for transfers to and from the toilet.

Used for users with good upper body & head control.

Used for transfers from a sitting or semi-sitting position.

SWL	275 kg	Washing	
Type	Half sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL

Low Back



Scan QR code for video - when to use Low Back sling



Model	Item no.
● Low Back XS	009-09633
● Low Back S	009-09634
● Low Back M	009-09635
● Low Back L	009-09636
● Low Back XL	009-09637
● Low Back XXL	009-09638

When to use


The sling provides excellent support under the thigh, and from the pelvis to the shoulder blade.

Used for users with a good upper body & head control.

Used for transfers from a sitting or semi-sitting position.

SWL	275 kg	Washing	
Type	Half sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XS - XXL

Leg sling

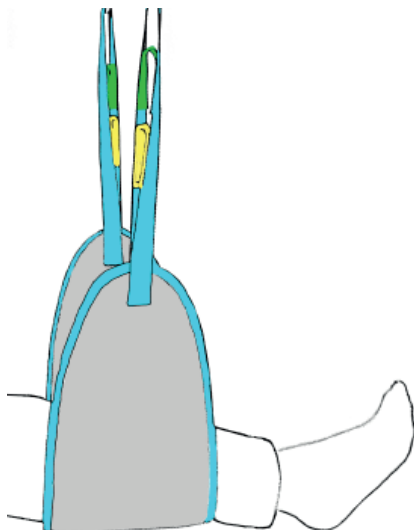

Model	Item no.
Leg sling, one size	008-04500

When to use

The leg sling is suitable for lifting and supporting legs or arms during personal hygiene, wound treatment or when getting the user dressed.

The sling is also suitable for positioning a full sling or putting on therapeutic socks, in case the user is not capable of lifting his or her leg or arm.

SWL	150 kg	Washing	
Type	Leg sling	Size	One size
Material type	Polyester		

Walking




Model	Item no.
● Walking XXS	009-09572
● Walking XS	009-09883
● Walking S	009-09884
● Walking M	009-09885
● Walking L	009-09886
● Walking XL	009-09887

When to use

The sling ensures safety and support, in case the user loses their balances during training.

The sling allows various adjustment options. It is possible to adjust the fit around the stomach area, the length of the leg straps and shoulder straps.

Used for training with Ergo Trainer or stationary hoists.
For users who need support with gait training and standing training.
Must NOT be used for lifting, since it is only tested for supporting and preventing the user from falling.

SWL	180 kg	Washing	
Type	Walking sling	Spreader bar	2, 3 and 4 point
Material type	Polyester	Sizes	XXS - XL



Scan QR code for **video** - when to use Walking sling



Standing



Model	Item no.
● Standing sling S	009-09594
● Standing sling M	009-09595
● Standing sling L	009-09596
● Standing sling XL	009-09597
● Standing sling XXL	009-09598

i When to use

The sling provides good support around the lower back, giving the user a sense of security when lifted and when standing.

For users with good upper body control and some lower limb function.

Used for transfers from a sitting position.

SWL	250 kg	Washing	85°
Type	Standing sling	Spreader bar	Standard
Material type	Polyester	Sizes	S - XXL



Standing 2-belt



Model	Item no.
● Standing sling 2-belt XS	009-09563
● Standing sling 2-belt S	009-09564
● Standing sling 2-belt M	009-09565
● Standing sling 2-belt L	009-09566
● Standing sling 2-belt XL	009-09567
● Standing sling 2-belt XXL	009-09568

i When to use


The sling with two belts is suitable for users with large stomachs as the belts can be separated and tightened over and under the belly.

The sling provides good support around the lower back, giving the user a sense of security when lifted and when standing.

For users with good upper body control and some lower limb function.



Used for transfers from a sitting position.

SWL	250 kg	Washing	85°
Type	Standing sling	Spreader bar	Standard
Material type	Polyester	Sizes	XS - XXL

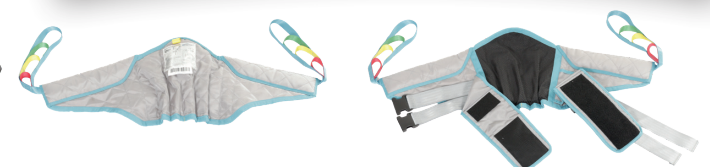

Scan QR code for **video** - when to use Standing sling



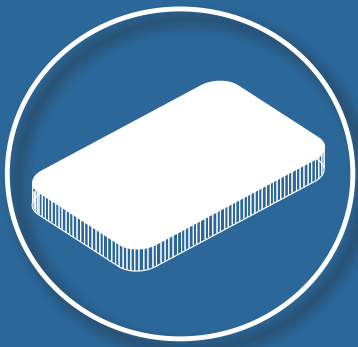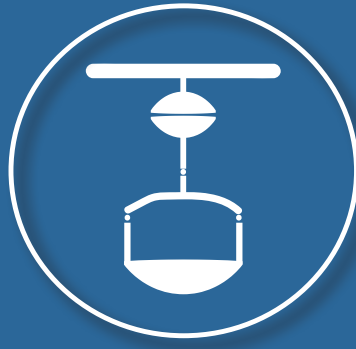

Do you have questions about one or more products in this brochure or would you like to know more about Winncare Nordic's solutions for patient lifting, hygiene, beds, mattresses and cushions, please feel free to contact us at any time.

Tel.: +45 7027 3720 / info@winncare.dk / www.winncare.com

WINCARE
N o r d i c