



UNIVERSAL COMFORT

Padded sling with extra support around the thighs and pelvis.

USAGE

- For users with increased disability or other special needs.
- For transfers from a sitting or lying position.
- For transfers in mobile and stationary hoists.

FEATURES

- The sling is transferring user from a wheelchair/chair or from lying position from a bed or bath.
- The sling provides good support of the body and stabilizes the pelvis, supporting the upper body and giving the user a feeling of comfort and safety.
- The sling has split leg straps, which do not cross, due to an extra closing strap. The leg straps have a large padded surface, which reduces the pressure on the entire femur.
- The sling ensures a good upright posture that facilitates the positioning of the user.
- Sling in polyester mesh is suitable for bathing.
- The sling has a small seat opening.

SPECIFICATIONS

SWL: 275 kg

Material:

Polyester and polyester mesh.

Sizes: XS-S-M-L-XL-XXL

ITEM NUMBERS:

Universal Comfort, polyester:

	Item no.
XS	009-09843
S	009-09844
M	009-09845
L	009-09846
XL	009-09847
XXL	009-09848

Universal Comfort, polyester mesh:

	Item no.
XS	009-09853
S	009-09854
M	009-09855
L	009-09856
XL	009-09857
XXL	009-09858




SLING SIZE GUIDE

The sling has a coloured strap on upper back with size and midline mark, which ensures correct placement of the sling.

Colour code is presented on the size chart.

The image below shows a yellow colour code, which means that the sling is in size M (Medium):

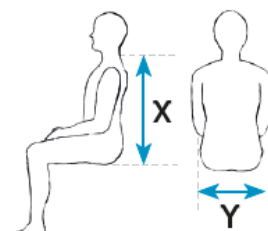

	XS	S	M	L	XL	XXL
Y (hip measurement)	37 cm	42 cm	48 cm	52 cm	68 cm	78 cm
X (measurement from neck to lower back)	50 cm	55 cm	66 cm	75 cm	75 cm	75 cm



TIP!

By placing the leg straps as illustrated below, the sling can also be used for amputee patients:



Please consult the technical manual for further specifications. All figures and measurements are for guidance only and may differ in actual situations. WinnCare Nordic reserves the right to changes without notice. Product photos may include accessories. Please consult your dealer for further information.

FOR MORE INFORMATION VISIT

winnCare.com

E-mail: info@winnCare.dk

Phone: +45 70273720

