

Luna Twin

From 275 to 400 kg

- simple and easy conversion

Luna Twin is a combi lift, consisting of two 2 single Luna units (each with 275 lifting capacity) and when combining the two units, you get the Luna Twin with a lifting capacity of 400 kg.

With Luna Twin, you only have to keep one type of lift in stock, no matter if there is need for a standard (275 kg) or bariatric (400 kg) lift.

Easy installation - no tools needed:

1. Attach the distance rod between the top of the units. Then mount the Luna units on the track and make sure that both straps are equal in length and connect the twin cable to the units.

2. Then attach the distance bar to the straps between the two Luna units

3. Attach the specially designed 2-point spreader bar with patented hooks to the distance bar.

The Luna Twin is controlled by a hand control (connected to one of the units). Adjust the straps by using the hand control (also during transfer, if needed).

The 2-point spreader bar can easily be replaced with the flexible 4-point spreader bar with a friction lock function, which makes sure that the spreader bar does not rotate unintentionally during the transfer.

The 4-point spreader bar is available in three sizes: 36 cm, 48 cm og 58 cm.

It is also possible to attach a spreader bar with weigh scale to the Luna Twin.

Key standard functions

- Weight capacity of 275 and 400 kg
- Spreader bar with patented hooks
- Soft start and stop function
- Emergency lowering (both powered and manual)
- Luna units and hand control IPx4 classified



2 x Luna
Twin
275 kg



Luna
Twin
400 kg

From Single to Twin
From 275 to 400 kg.
5 minute installation. No tools.



Model	Item number
Luna Twin 275 kg (1 unit)	015-03040
Luna Twin 400 kg (2 units) incl. distance rod and distance bar	015-03042
Accessories *	Item number
Hand control IPx4	015-03120
Spreader bar for Luna Twin	015-02202
Spreader bar 36 cm, 4-point	015-02036
Spreader bar 48 cm, 4-point	015-02037
Spreader bar 58 cm, 4-point	015-02038
Distance rod for Luna Twin	016-02202
Distance bar for Luna Twin	015-02200

* See separate brochure for hand control, spreader bars, weigh scale and slings.

TECHNICAL SPECIFICATION

	Single	Twin
Max. lifting capacity	275 kg	400 kg
Strap length	2200 mm	2200 mm
Weight	6,7 kg without spreader bar	2 x 6,7 kg without spreader bar
Length/height	320 x 130 mm	720 x 130 mm
Batteries	36 V, lead/acid	36 V, lead/acid
Charging	100- 240 V, 0,5 A	100- 240 V, 0,5 A
Chassis	High impact, moulded ABS	High impact, moulded ABS
Noise (LWA)	During lifting/lowering without load: 48dB(A)	During lifting/lowering without load: 48dB(A)
Noise (LWA) max load	During lifting/lowering: 52dB(A)	During lifting/lowering: 52dB(A)
Full charging time	4,5 hours	4,5 hours
Lifting speed	39 mm/sec. with 90 kg weight	39 mm/sec. with 90 kg weight
Lifts per charge:	80 lifts	80 lifts
Protection class	IPx4	IPx4

- Soft start / stop
- Battery status indicated by light and sound
- Beskyttelse mod overbelastning
- Emergency stop via hand control



All protection class tests performed by TÜV.



With Luna Twin room to room transfer is also possible. Just slide the Luna Twin lift through the unique door connector from Ergolet .

Consult the Technical manual for further specifications. All figures and measurements are for guidance only and may differ in actual situations. WinnCare reserves the right to changes without notice.

Product photos may include accessories. If you have any questions, please consult your dealer.

CE MDD93/42/EEC
 Class I

date 18.01.2022 - ver.1

WinnCare
 Taarnborgvej 12C
 4220 Korsoer
 Denmark
 tel. +45 70 27 37 20
 e-mail: info@winnCare.dk

www.winnCare.com

WIN CARE